

**[Easy extraction manual]**

1. <EASY PERIOTOME> Using periostome, cut the coronal circumferential PDL of the root
2. <EASY PERIOTOME> Insert periostome mesially and apply gentle malleting. Feeling dislodgement of the root, insert the periostome apically.
3. <EASY PERIOTOME> If there is resistance, remove the periostome by oscillating towards the lingual side (Important note: If too much force is applied, the instrument can fracture.)
4. <EASY PERIOTOME> Insert periostome distally applying malleting and enter towards apex of the root.
5. <EASY PERIOTOME> If enough dislodgement of the root is felt, remove the periostome.
6. <EASY PERIOTOME> If there is not enough dislodgement, perform further malleting mesially and distally.
7. <EASY LUXATOR> Select luxator which best fits the root and perform luxation either mesially or distally. If there is enough dislodgement from EASY PERIOTOME, this step can be skipped.
8. <EASY FORCEP> Gripping onto the subgingival root with the forceps, deliver the root by applying bucco-lingual force and rotation movement.

[Precautions]

1. Excessive flexing of the periostome within the root can lead to fracture of the periostome.
2. When malleting, 3mm or larger EASY PERIOTOME need to be used. This set includes 3mm periostome.
3. If more force than the luxator is required, elevator will need to be used.